







INSTRUCTIONS FOR USE



- ① Ring-pull Cap
- ② Body of the Container
- ③ Hook

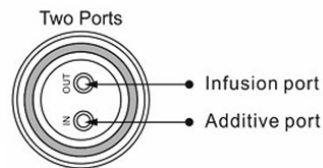

For more Information

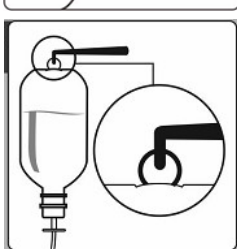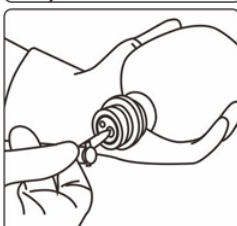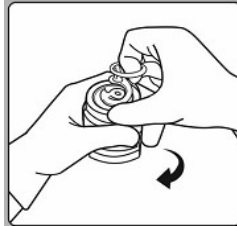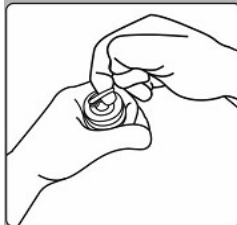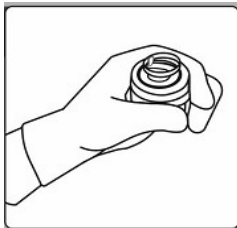

Administration without Medication Additives

1. Hold the stopper with left hand
2. Insert right hand index finger into the ring on the cap of the resealable rubber stopper
3. Pull and twist clockwise to remove the cap
4. Hold the bag with the ports facing towards the operator.
5. Insert the spike of the infusion set into the OUT port of the rubber stopper until it is firmly seated.
6. Attach administration set.
7. Hang the bag on the IV pole with the hook at the bottom of the bag.

Administration with Medication Additives

1. Hold the stopper with left hand.
 - a. If adding medication during solution administration: close clamp on the set, remove bag from IV hook, set aside and skip to step 4.
2. Insert right hand index finger into the ring on the cap of the resealable rubber stopper
3. Pull and twist clockwise to remove the cap and set aside
4. Using a syringe with 18 gauge \times 1 1/2 (1.2mm \times 40mm), BD blunt fill needle-filter, prepare medication
5. Hold the bag with the ports facing towards the operator.
6. Puncture the IN port of the resealable rubber stopper with the syringe needle and inject.
7. Mix solution and medication thoroughly.
8. Attach administration set.
9. Hang the bag on the IV pole with the hook at the bottom of the bag.